

WEB TORSO RESTRAINT

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING



INTENDED USE: To secure a person's waist while in bed. It can be used with or without wrist and ankle restraints. Use only as specified in instructions.



INSTRUCTIONS: Unlock all locks and lay the restraint flat on the bed. Each side should have a lock near the edge of the bed and a third lock should be in the middle of the bed. Attach the side straps to the bed by looping the end of the strap around the bed frame and pull through the lock. Lock by using instructions on back of this page. Position the person on the restraint so the



waist belt will be comfortable when secure. Bring both ends of the waist belt together around the person and secure by looping the tail end through the lock. Continue by following locking procedures on back of this page.

NOTE: Do not use for prolonged periods of time.

INVENTORY ID:

	Waist Size Range	Approx. Weight
30L4S	24-36"	8 oz
30L4M	30-42"	8 oz
30L4L	36-48"	8 oz

DESCRIPTION: The side straps from lock to end are 40" long and 2" wide.

SAFETY CHECK: Including, but not limited to, locks, stitching, and polypropylene webbing.

MATERIALS USED: Humane Restraint L-400 locks and 2" polypropylene web.

CLEANING AND MAINTAINING: Dry clean or launder in your facility's laundry at less than 280°F. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/3/05