

POLYURETHANE BELTS–NON-LOCKING

IMPORTANT PRODUCT INFORMATION—PLEASE READ BEFORE USING

INTENDED USE: Including, but not limited to, securing our wrist and ankle restraints.



Wrist and ankle restraints sold separately.



INVENTORY ID:

ITEM	LENGTH	APPX. WEIGHT
B-218-POLY	18"	5 oz
B-224-POLY	24"	5 oz
B-230-POLY	30"	7 oz
B-236-POLY	36"	8 oz
B-242-POLY	42"	8 oz
B-248-POLY	48"	9 oz
B-254-POLY	54"	10 oz
B-260-POLY	60"	11 oz
B-266-POLY	66"	12 oz
B-272-POLY	72"	13 oz
B-278-POLY	78"	13 oz

INSTRUCTIONS: Feed the tail through the roller buckle to desired position and secure tail with the keeper. If using with a restraint, feed the tail through the restraint loop before feeding into the roller buckle.

DESCRIPTION: Belts are 1 1/4" wide by specified length. The 18" belts have five size adjustments, 24" belts have ten size adjustments, and all other lengths have 12 size adjustments.

SAFETY CHECK: Including, but not limited to, rivets, polyurethane, and roller buckles.

MATERIALS USED: 1 1/4" polyurethane, 1 1/4" steel roller buckle, and 1 1/4" steel keeper.

CLEANING AND MAINTAINING: For regular cleaning, wipe with a mild soap and warm water. When decontaminating bloodborne pathogens, use one cup of bleach to a gallon of water, then rinse and dry.

STORAGE: Store away from heat greater than 280°F or open flame.

Caution: Federal law restricts this device to sale by or on the order of a physician.

Rev 2/3/05