

**AMD Technologies, Inc.**

218 Bronwood Avenue
Los Angeles, CA 90049-3104
USA

T 310.471.8900
C 310 350.3657

W amdtechnologies.com
E matthew@amdtechnologies.com

**Imaging
Accessories**

STANDARD TABLE PADS



DESCRIPTION

AMD Technologies, Inc.'s Standard Table Pads feature a durable grey vinyl cover over a supportive foam core, offering a cost-effective solution for added patient comfort during imaging procedures. Lightweight and easy to handle, these pads are designed for use on imaging tables as well as grommets for quick storage.

The radiolucent construction allows for clear imaging without interference, while the smooth surface can be easily cleaned with standard mild cleaners or disinfectants.

Available in standard sizes: 24" x 72" x 1" and 30" x 80" x 1"



CLEANING & CARE INSTRUCTIONS FOR TABLE PADS

To maintain the quality and longevity of your foam products, please follow the guidelines below for proper cleaning, care and storage.

ROUTINE CLEANING

To remove dirt or dust from table pads and vinyl bolsters, clean the surface with cold water and a mild general-purpose soap.

Avoid detergents containing hydrocarbon solvents such as acetone or ammonia.

Do not use abrasive pads, stiff brushes, or aggressive scrubbing, as these may damage the surface.

DISINFECTION

For surfaces exposed to blood, bodily fluids, barium, betadine, or other contaminants, use hospital-approved disinfectant wipes or sprays.

Do not autoclave, steam, soak, launder, or submerge the product in water.

CARE

Table pads are designed to provide patient comfort and support during imaging procedures while offering a cleanable surface between uses.

If the surface becomes punctured or torn, it can no longer serve as an effective infection control barrier. In such cases, the product should be removed from service immediately and replaced.

STORAGE

Store table pads on a flat surface to prevent creasing or hang them on an appropriate table pad rack.