

McKESSON

Foley Catheter

100% SILICONE | 5-10cc BALLOON

STRAIGHT TIP | 2-WAY | UNIVERSAL | STERILE

16 in | 18 Fr | 1 | 
PER PACK | DO NOT REUSE

INSTRUCTIONS FOR USE

1. Follow preferred aseptic procedure for catheterizing patient.
 2. To inflate retention balloon, insert syringe tip tightly into needleless valve and introduce appropriate volume of water:
5-10cc balloon uses 10mL of sterile water. DO NOT INFLATE OVER 15mL.
30cc balloon uses 35mL of sterile water. DO NOT INFLATE OVER 40mL.
Note: Do not exceed recommended capacities.
 3. To deflate retention balloon, gently insert a syringe in the catheter valve. Never use more force than required to make the syringe “stick” in the valve. If you notice slow or no deflation, reseal the syringe gently. Allow the balloon to deflate slowly on its own. Do not aspirate or manually accelerate the deflation of the balloon.
- Valve Type: Use luer slip tip syringe. Do not use needle.

WARNING: Do not use petroleum-based ointments or lubricants on catheter. This will damage latex and may cause the balloon to burst. Dispose of appropriately after procedure. To help reduce the potential risk of infection and/or other complications such as allergic reaction or inaccurate flow rates, do not reuse.

CAUTION: For urological use only. Do not aspirate urine through drainage funnel wall. Federal (USA) law restricts this device to sale by or on the order of a physician.

Avoid storage in direct sunlight/fluorescent lighting and store in a cool, dry, and well-ventilated area.

Contents STERILE in unopened, undamaged package. Do not resterilize.

R_x ONLY

Questions? Call 1-800-777-4908

Distributed by McKesson Medical-Surgical Inc.

Richmond, VA 23233

PVN B1223

Made in China

STERILE | EO

MFR # **16-SFOL518**

